

ICE PACK 160g 45146 TECHNICAL DATA SHEET



DESCRIPTION:

The ice pack can decrease the temperature rapidly and last for 40 minutes normally.

It is no harm to human and the environment, which can also be used on beauty of face, eyes etc.

INTENDED USES:

It is designed to reduce the temperature.

It reduces pain and expedites recovery for sprains, slight bumps and dislocation of bone joints.

This ice pack can lower temperature from fevers, relieve pain from toothache and itch from insect bites.

DIRECTIONS FOR USE:

1. Locate inner liquid pouch, squeeze firmly to rupture.
2. Shake to mix contents, speed up chemical reaction.
3. The endothermic reaction, make the product quickly cooling.
4. Apply to patient.
5. Discard it after use.

CAUTION FOR USE:

The product is for single-use only. If the package is broken, please do not use.

In case of too lower temperature, please use towels or cotton to cover it firstly.

Do not puncture the ice pack, if the package break and fluid contact the skin or eyes,

Please flush area with large amounts of water immediately.

Do not swallow the contents, if the contents have been swallowed accidentally,

Please drink large amounts of water firstly and contact poison control center or physician immediately.

Cold therapy should not be used by individuals with circulatory problems,

Unless under the direct supervision of a physician.

For external use only. Adult supervision recommended.

CE CLASS:

Class IIa

PHYSICAL PROPERTIES:

Package Material: PA(Polyamide) + PE(Polyethylene)

Inner Liquid Bag Material: LDPE

Ingredients: Water (CAS#: 7732-18-5), Carbamide (CAS#: 57-13-6)

Weight:160g

STANDARD SIZE/PACKAGING:

Item number	PCS/Carton	Carton Size
45146	160	46*31*23

*Customized shapes/sizes could be available upon request.

STORAGE:

The above product should be stored in a cool dry environment for best results of product performance.

Keep away from light and heat (store below 30°C)

SHELF LIFE:

Best before 5 Years

LEGAL MANUFACTURER:

RFX+CARE Manufacturing Co.,Ltd.

7 Lanjiang Road, Economic Development Zone,

Shaoxing, Zhejiang, China 312000.

CERTIFICATES TÜV SÜD:

EN ISO 13485:2012

REFERENCE REPORTS AND CERTIFICATES:

Annex.1 SGS RoHs Test Report

● You are welcome to ask for the above mentioned test report. Please contact your local distributor
Or send your request to email info@rfx-care.com for more information.

Testing represents typical performance values (averages). Technical information provided by
RFX+CARE Manufacturing Co.,Ltd. is based on experience and/or tests believed to be reliable, but their
Accuracy is not guaranteed and the results may not be relevant to every user's application.

For this reason RFX+CARE Manufacturing Co.,Ltd. does not accept responsibility or liability, direct
Or consequential, arising from reliance upon any information provided and the user should determine
The suitability of the products for their intended use.

Oppenheim & Jansson

Oppenheim & Jansson A/S

Bakkegaardsvej 408, DK-3050 Humlebaek, Tel:+45 45 76 90 20, Fax: +45 45 76 02 98

www.oppenheim-jansson.com - e-mail: info@oppenheim-jansson.com